

Quick guide for importing Elos Accurate[®] Library



Elos Accurate[®] Library

December 2017

Content:

- Download the latest Elos Accurate[®] Library 2
- Before installing new libraries 3
- Importing the Elos Accurate[®] Library in to DWOS 6

Elos Accurate[®] Library

Download the latest Elos Accurate[®] Library

To download the latest Elos Accurate Library, visit the Elos Accurate web site at the following link
<http://elosdental.com/support-training/downloads/>

By signing up for the Elos Accurate newsletter you will always be informed about new Elos Accurate Library releases. Sign up at <http://elosdental.com/>.

The Elos Accurate Library offers a complete digital open workflow solution for Desktop or Intra Oral scanning.

The Elos Accurate libraries are divided into groups to make it easy to download the relevant library or all.

Elos Accurate[®] Hybrid Base[™] Bridge: Contains prefabricated components with a titanium interface towards the implant with engaging interface. It is used for screw-retained bar or bridge constructions.

Elos Accurate[®] Hybrid Base[™] Kit: Contains prefabricated components with a titanium interface towards the implant with engaging interface. It is used for screwed single or small cemented bridge restorations.

Single Abutment: Is used for milled single abutments using the Elos Pre-milled Abutments.

Bar & Bridge: Is used for creating a bar or bridge in one piece and includes the interface geometries.



Elos Accurate[®] Library


Before installing new libraries

We recommend to always remove old libraries before installation of the new Elos Accurate[®] Library. If this is the first time an Elos Accurate[®] Library is installed, you can proceed to the next chapter of this instruction.

- Start the DWOS Client.




This opens the DWOS Client software. Click on the **Settings** icon  and select **Implant Kits**

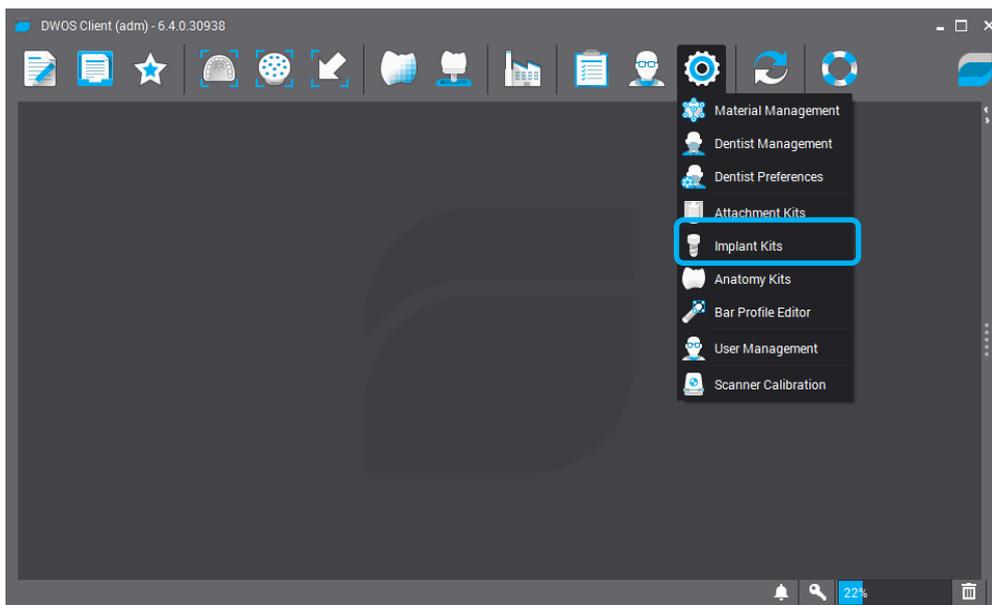


Elos Accurate® Library

Now the Implant Kit Editor tab appears in the upper left corner of the DWOS Client.

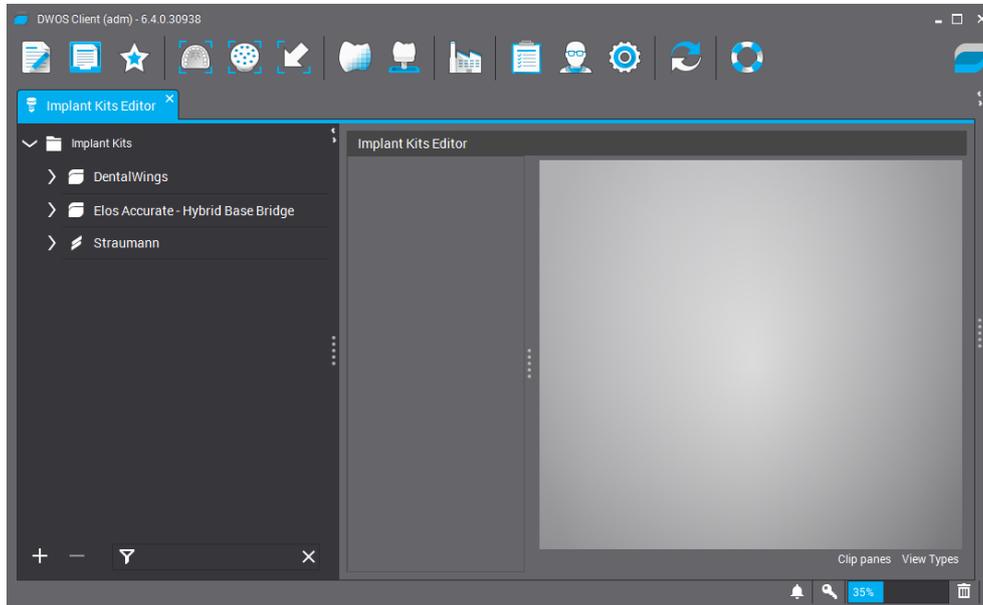


There are two ways to remove libraries from DWOS.

First option: Click on the library name that you want to delete to mark it. Click on the “**minus**” sign in the bottom to delete the library.

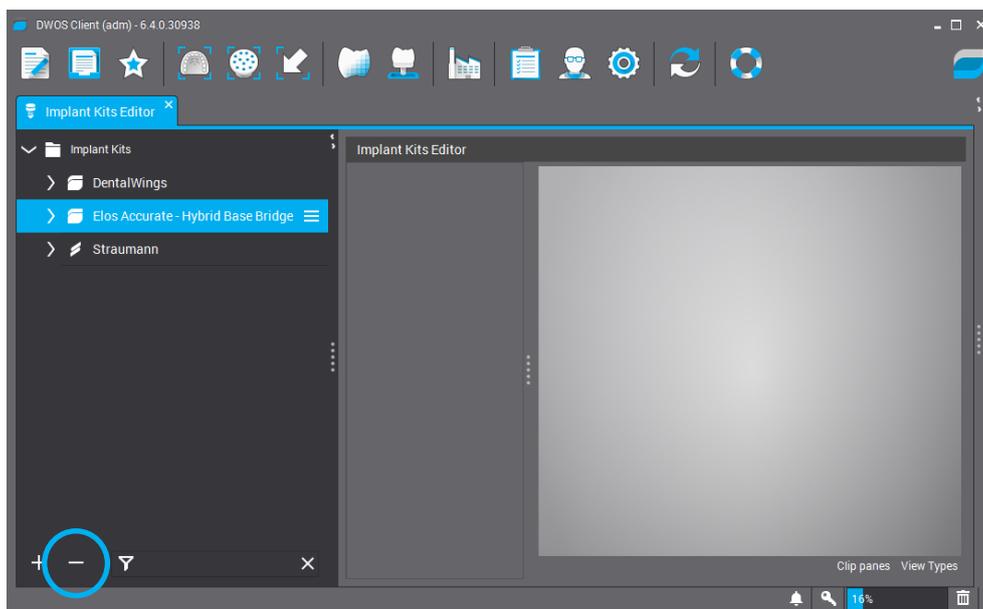


Elos Accurate® Library

Second option: Click on the library name to mark it. Click on the menu icon next to the library name, and chose **“Delete Manufacturer”**

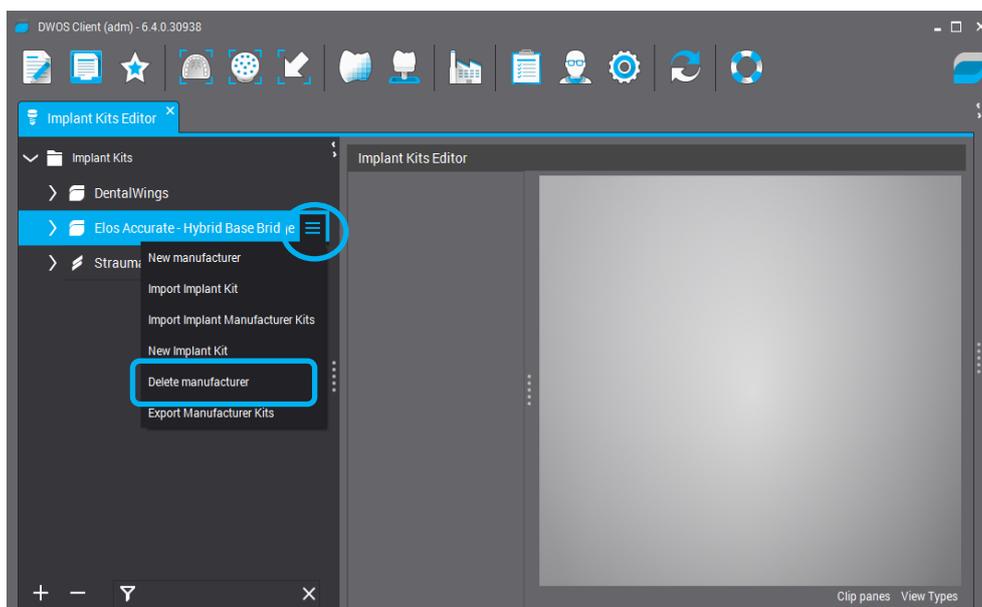


Click OK to remove and the library has now been deleted.

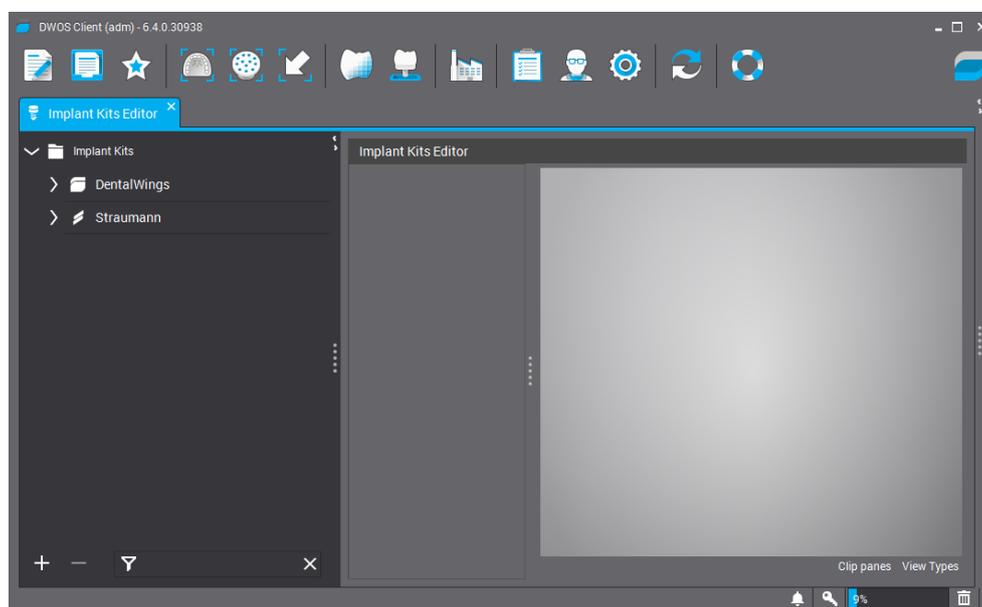


Elos Accurate[®] Library

Importing the Elos Accurate[®] Library in to DWOS

To install the Elos Accurate[®] Library in to DWOS simply click on the “Plus” sign in the lower left corner of the DWOS Client, and click on “**Import Implant Manufacturer Kits**” to import the full library.

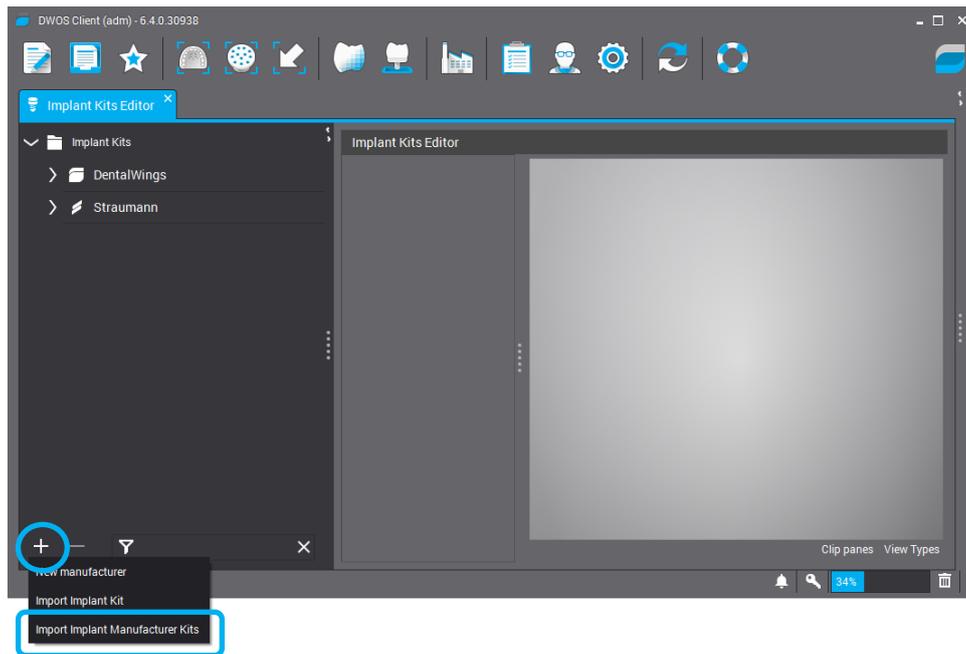

A pop-up window appears. Locate the downloaded libraries, and chose the library to install. Press “**Open**” to begin installation.

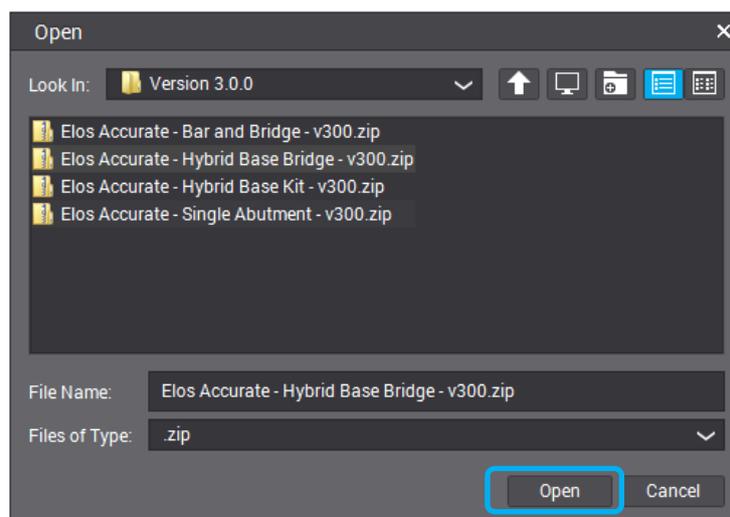

Elos Accurate[®] Library

Now the Elos Accurate[®] Library is installed and ready to use, repeat the steps for all the libraries you wish to install.

